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
EN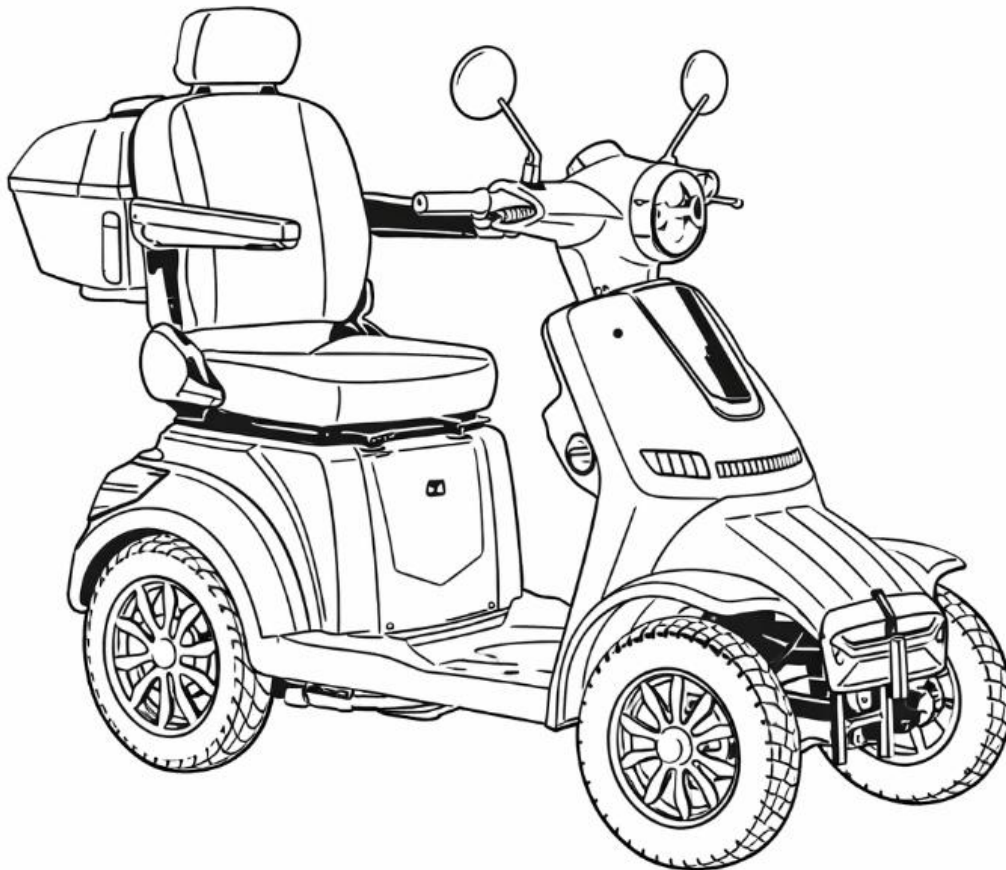

The Agilon Elan mobility scooter is designed especially for seniors and persons with reduced mobility to facilitate movement and transportation.

The device has been designed to be as safe and easy to use as possible. To ensure safe and proper use, please read this manual carefully and follow the instructions provided.

Keep this user manual for future reference

Due to continuous product development, the text, images and other content of this manual may differ slightly from your device. The manufacturer reserves the right to make changes to the device. We appreciate your understanding and ask that you always primarily follow the actual features of the device.

1. Safety information


Read the safety instructions carefully before using the electric scooter. Ensure that you are familiar with local road traffic laws and related regulations. Always take into account the prevailing weather conditions and driving environment when operating the device.

1. Before driving on public roads, practice using the mobility scooter in a safe area. Pay particular attention to practicing the use of the controls, braking and turning.
2. Always inspect the vehicle before starting.
 - a. Ensure that the battery is sufficiently charged.
 - b. Check the scooter tires for external damage or loss of pressure.
3. Take into account the environment and driving conditions while operating the device.
 - a. Reduce speed if there are many people or other vehicles around you.
 - b. Adjust your speed to the conditions and drive at a moderate speed, for example on slopes and uneven surfaces.
 - c. Continuously monitor your surroundings and road conditions.
 - d. Before entering traffic, test that the brakes and control devices operate as expected.
4. Approach thresholds, curbs, slopes or other obstacles head-on at a low speed.
5. Do not touch the wheels or other moving parts while the device is in operation and ensure that clothing or other objects cannot become entangled in them.
6. Keep the parking brake engaged at all times when stationary.
7. Mount and dismount the device on a level surface.
8. Keep both hands on the handlebars and your feet in the designated area while driving.
9. Always adjust your speed to the conditions and reduce speed when turning and when driving on inclined surfaces.
 - a. Be aware that braking distance increases when driving downhill. Anticipate the need to slow down or stop and begin braking in good time.
 - b. Reduce speed before turning, exercise caution and pay attention to other traffic at intersections.
 - c. Observe the road surface and aim to detect potholes and other potential obstacles in sufficient time.
 - d. Take into account other road users such as drivers and pedestrians.
10. Braking distance increases on slippery surfaces. Slippery conditions may also affect steering control in turns and overall tire grip. Maintain a moderate speed and avoid unnecessary driving in slippery conditions.
11. Avoid rapid acceleration and situations that may require sudden braking.
12. Do not allow other persons to use the device if they are not familiar with its operation.
13. Do not allow children or persons with impaired judgment to use the device.
14. The device is designed for one person only, do not attempt to carry passengers.
15. Wear a helmet when driving. We recommend suitable eyewear and other protective equipment.
16. Use headlights and adequate reflectors when traveling in low light or darkness. Before setting off, ensure that you are able to observe your surroundings and driving conditions even in poor lighting. Ensure that other road users are able to see you.
17. Do not operate the device under the influence of alcohol, narcotics or medications that may impair driving ability.

2. Device components and functions


Scooter components

1. Rearview mirror
2. Headlights
3. Turn signal
4. Tires
5. Aluminum rim
6. Brake disc
7. Throttle grip
8. Brake lever
9. Seat
10. Ignition switch
11. Armrest
12. Storage box
13. Rear light
14. Charging port
15. Anti-tip wheel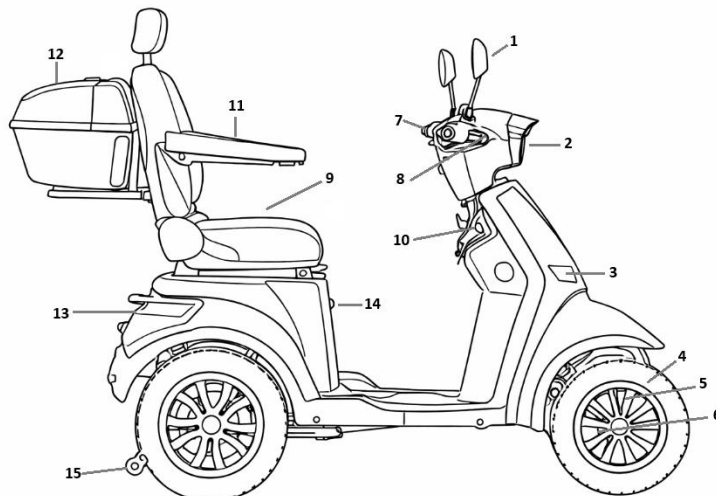

Control unit

1. High/low beam switch
2. Turn signal
3. Horn
4. Display panel
5. Hazard warning lights
6. Light switch
7. Parking gear
8. Drive gear
9. Reverse gear
10. Ignition switch

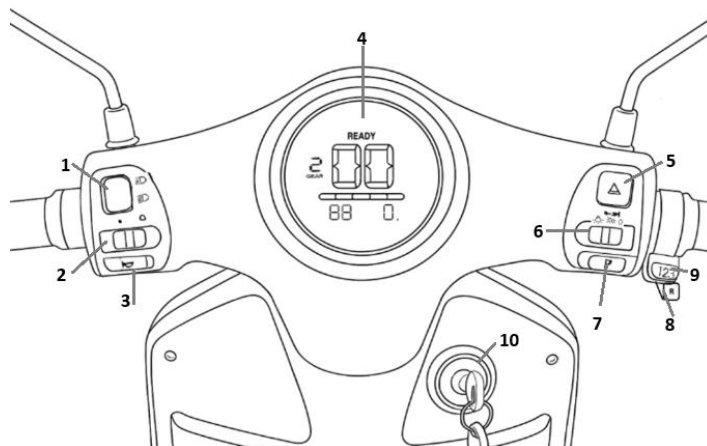Display

1. Gear (1, 2, 3)
2. Speedometer (km/h)
3. Battery voltage (V)
4. Odometer (km)
5. Battery level indicator

Display symbols

1. Turn signal
2. High beam
3. Ready mode
4. Daytime running lights
5. Reverse gear
6. Warning symbols
 - a. Motor indicator
 - b. Controller indicator
 - c. Brake indicator
 - d. Throttle indicator
7. Drive gear

At the bottom of the display there is an odometer which, upon startup, shows the total distance driven with the scooter for 5 seconds. After this, the odometer begins measuring a new trip.

4. Usage

4.1 Starting

Start the scooter by turning the key in the ignition switch clockwise to the “ON” position.

The display indicators will light up and flash briefly, and the text “READY” will appear in green at the top of the display.

4.2 Gears

The scooter has three drive gears as well as reverse gear, parking gear and neutral mode. The active gear is shown on the left side of the display above the text “GEAR”. The reverse gear indicator is shown separately on the right side of the speed display.


Driving gears (1,2,3)

The desired drive gear is selected using the rocker switch according to the driving speed.

1 = maximum speed 10 km/h

2 = maximum speed 15 km/h

3 = maximum speed 25 km/h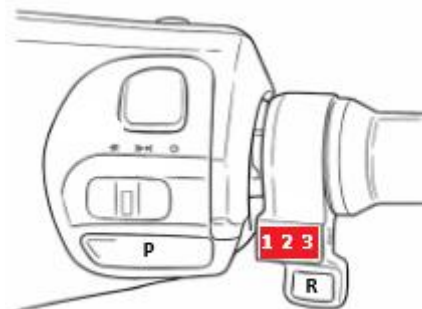

Reverse gear (R)

The reverse gear is activated and deactivated using the “R” button located on the lower part of the right handlebar. When reverse gear is active, a yellow “R” indicator appears at the top right of the speed display.

Parking gear (P)

The parking gear is activated using the P button or a separate hand brake lever. When the parking gear is activated, the letter P appears on the speed display.

When the parking gear is active, the scooter’s parking brake is engaged. Always use the P gear when parking or getting on or off the scooter.

Neutral

In neutral mode, the parking brake is disengaged and the scooter does not drive or brake with the motor, allowing it to be moved manually if necessary. Release the parking brake only when you need to move the scooter by pushing or guiding it. Ensure that you are always able to activate the brake using the lever and stop the scooter to avoid hazardous situations.

4.3. Brake and throttle

Scooter brakes are activated by pulling the brake handle. The left brake lever activates the rear brake and the right brake lever activates the front brake. Practice using the brakes in a safe environment at different speeds before driving in traffic.


The parking brake is activated using a separate “P” button. Pulling and releasing the brake lever releases the parking brake.

The scooter throttle operates by twisting the grip. Before starting, ensure that you have selected a suitable drive gear (1, 2, 3) and practice using the throttle in a safe environment before driving in traffic.

4.4. Turn signals

The scooter turn signals are operated using the button on the left side of the control unit.


Use turn signals to indicate turning or a change of direction. Activate the turn signal well in advance of turning and ensure that other road users can notice it. When the turn signal is active, green flashing arrows are shown on the display. Remember to turn off the signal after completing the turn.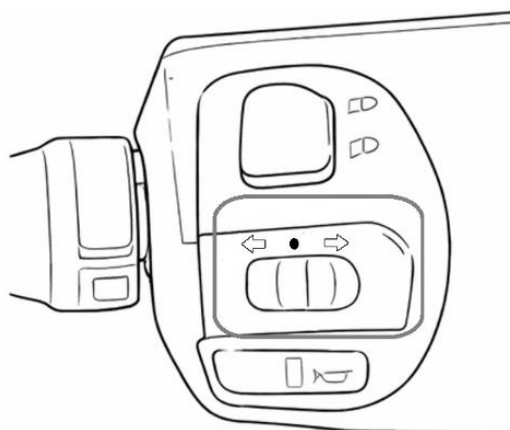

4.5. Headlights


The headlights are switched on using the switch located on the right side of the control unit.

-  Lights off
-  Parking lights on
-  Headlights on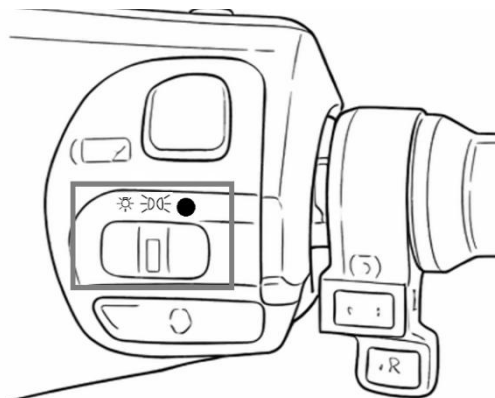

The headlights have two functions, normal driving lights and high beam. The light setting is adjusted using the switch on the left side of the control unit. High beam can be used to increase visibility when driving in the dark. High beam should not be used when driving in daylight.


-  Normal headlights
-  High beam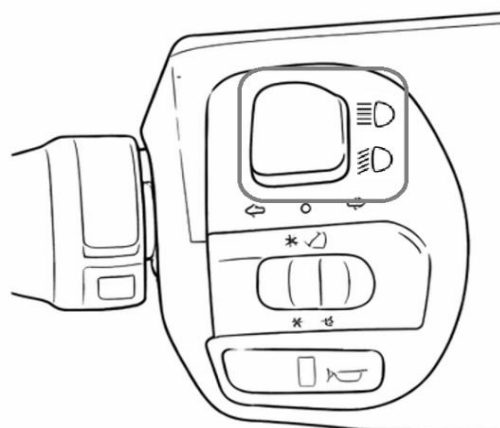

4.6. Hazard warning lights and horn


The scooter horn is located on the left side of the control unit.

Use the horn to warn other road users or to draw attention in a hazardous situation.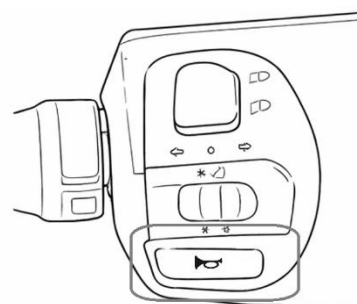

The vehicle hazard warning lights are activated using the button on the right side of the control unit.

Hazard warning lights are used when the scooter has stopped unexpectedly or is moving unusually slowly and you want to warn others. Turn off the hazard warning lights when the situation is over.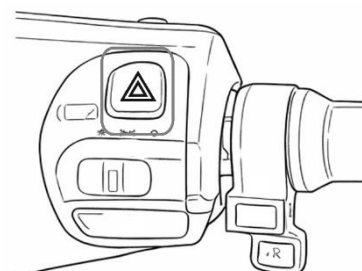

5. Driving

5.1 Before starting

Check the following before starting:

Tires

Check that the tires are properly inflated and that nothing is stuck in them that could cause a puncture. Check that the tires have sufficient tread and that there are no visible cracks or other damage. When starting, ensure that the tires rotate normally and that no abnormal sounds are heard while rotating.

Battery

Check that the battery has sufficient charge. The battery level is indicated by bars on the display below the speed. In addition, the battery status is indicated by the V value shown in the lower left corner of the display.

Lights and horn

Check that the headlights turn on and off normally. Check that the horn button functions and that the horn sound is clearly audible.

Control unit

Ensure that the throttle responds normally and does not feel loose. Test that the brakes function properly. Check that the indicator lights on the display are working.

5.2 Driving

Driving forward

1. First select the drive gear and disengage the parking gear
 - a. Gears 1, 2 and 3: forward driving
 - b. R gear: reverse
2. Turn the throttle gently
3. The scooter will start moving

Braking

Release the throttle and pull the brake lever gently.

Practice braking first in a safe area. Ensure that you understand the required braking force when turning and parking before driving in traffic.

Turning

Reduce speed and turn the handlebar gently.

Practice turning first in a safe area. Ensure that you understand the required braking force when turning before driving in traffic. Sudden handlebar movements, turning at too high a speed and strong movements increase the risk of tipping over.

Consider driving conditions

- Approach thresholds carefully and avoid crossing obstacles higher than 5 cm. Always cross obstacles straight on.
- Avoid driving on steep slopes and note that driving on slopes affects speed and braking distance.
- Pay attention to surfaces that are sloped sideways, such as inclines, and exercise caution.
- Do not attempt to cross gaps in the road surface wider than 10 cm.
- Avoid driving on soft or low-friction surfaces such as loose sand or grass, snow or ice.

6. Battery and charging

6.1. Charging

Read the battery maintenance and safety instructions carefully before charging the battery.

1. Start charging by switching off the device.
2. Connect the charger power cable to the charger and plug it into a power outlet.
3. Connect the charger to the scooter charging port.
4. The charging indicator is shown on the scooter display
5. When charging is complete, disconnect the scooter from the charger and then unplug the charger from the power outlet.

Note that the scooter must not be stored connected to the charger for long periods after charging is complete.

6.2. Battery maintenance and safety

Read the operating and safety instructions for the device, battery and charging carefully.

1. Use only the original charger supplied with the device.
2. Charging conditions
 - a. Charge the device in an environment with a temperature of 0–40 °C. Avoid charging at high temperatures. High temperature may cause the battery to overheat and become damaged. Charging below 0 °C should be avoided to prevent battery overload.
 - b. Charge the battery only in a dry and sufficiently spacious area with good ventilation. Ensure that no flammable gases are stored in the charging area due to risk of explosion. Charging in a humid environment may cause a risk of short circuit. Ensure that the charging space is at least twice the size of the scooter.
 - c. Ensure that during charging the device or charger does not come into contact with textile surfaces such as upholstered furniture, bedding or curtains.

3. Charging requirements

- a. Charge the device using a 110–220 V power supply. Avoid excessively high or low voltage, as it may damage the device or cause hazardous situations.
 - b. The scooter battery typically requires 5–10 hours to reach full capacity. Do not keep the scooter connected after charging is complete. Keeping the charger connected for more than 10 hours may reduce battery quality.
 - c. Do not touch the battery or charger during charging or immediately after charging, as they become hot during charging and contact may cause burns.
 - d. Do not touch the battery or charger connection terminals. Do not wet the connection terminals and ensure that they are dry before starting charging. Do not allow the terminals to come into contact with metal objects or surfaces due to risk of short circuit.
 - e. Do not use extension cords for charging to prevent the risk of electric shock, short circuit or fire.
 - f. Do not pull the power cable forcefully to prevent the risk of electric shock, short circuit or fire.
 - g. Do not handle the battery with wet hands to avoid the risk of electric shock.
 - h. Ensure that the brake is engaged and that the device cannot move during charging. Do not move the scooter during charging. Movement while connected to the charger may cause an accident.
4. Do not attempt to disassemble the battery or charger. Disassembly may result in fire or explosion. The warranty is void if the battery, charger or mobility scooter is disassembled.
5. Do not start or charge the device if the battery has become wet or if water has entered the battery compartment.
6. Storage of battery and charger
- a. Keep the scooter and the attached battery at a sufficient distance from heat sources to prevent explosion or other accidents caused by overheating.
 - b. Keep the battery and charger dry, do not immerse them in water under any circumstances. Moisture may cause short circuit or other malfunctions.
 - c. Avoid impacts to the battery or charger (collisions, dropping or rough handling) to prevent accidents caused by damage to internal components.
 - d. If the battery is externally damaged or shows abnormal operation, stop using and charging it immediately. Check the charger and battery regularly for damage or wear. If problems are detected, replace the charger or battery immediately.
 - e. If the scooter is not used for at least 30 days, the battery must be charged to 80% (four bars on the display) and stored in a cool and dry place. During long-term storage, the device must be charged at least every 30 days for four hours to keep the charge level at a minimum of 50% (three bars on the display). Charge the device to 100% every three months to extend battery life.
 - f. If only one bar remains on the battery level display after use, the device must be charged within 7 days to prevent damage due to excessive discharge. Note that if the battery is not charged in time, it may be permanently damaged due to excessive discharge. Battery failure caused by excessive discharge is not covered by the warranty.

7. Maintenance and upkeep

To maintain the operational condition of the mobility scooter and to detect any maintenance needs in a timely manner, follow the instructions below.

Cleaning

Clean the scooter by wiping its surfaces regularly. Use mild cleaning agents and do not use abrasive or scratching cleaning tools or strongly alkaline or acidic cleaning agents.

Do not rinse the motor, battery, other electrical components or connectors with water. Avoid cleaning with running water or a pressure washer to prevent water from entering areas where it may damage the scooter's operation.

Tires

Check the tire surfaces regularly and pay attention to wear and any abnormalities.

Check the tire pressure regularly. The front and rear tire pressure should be 2.25 bar.

Battery and charging

Charge the battery regularly to prevent excessive discharge. If the device is not used, refer to the instructions for battery maintenance during storage in section 6.2 of this manual.

Storage

Store the Agilon Elan mobility scooter in a dry space with a temperature of 0–30 °C.

If the device is temporarily stored outdoors or used in damp conditions, protect at least the battery housing and control units from rain and direct sunlight.

8. Troubleshooting


Error or fault	Cause of disturbance	Troubleshooting
The device does not start when the key is turned in the ignition switch	The battery has no charge	Connect the device to a power source and ensure that the battery begins charging.
	The ignition switch is damaged	Contact the dealer or service to repair the faulty part.
The device does not respond when the throttle is turned	The gear is incorrectly selected	Ensure that the selected drive gear (1, 2, 3) or reverse gear (R) is shown in the GEAR field on the display.
	The battery voltage is too low	Fully charge the battery
The device does not charge even though it is connected to a power source	The charging connector or plug is not properly connected	Securely connect the plug and charging connector
	Power supply disturbance	Check with another device whether the outlet is supplying power
	The battery or charger is faulty	Contact the dealer or service
The device moves only slowly	Incorrect drive gear	Change to a drive gear suitable for higher speed
	The battery is almost empty	Charge the battery
Unusual noise during operation	Loose fasteners or components	Locate the source of the noise and tighten the components, contact service if necessary
	Foreign object in the tires	Check the tires and remove any object interfering with operation

8. Technical specifications

Model name	Agilon Elan
Motor	1000 W
Maximum speed	25 km/h
Range	50 km
Battery	60V 20Ah
Charging time	5-10 h
Tire size	Front: 8 inches Rear: 10 inches Tire width: 3 inches (approx. 76 mm)
Load capacity	170 kg
Device dimensions	Length: 1510 mm Width: 680 mm Height: 1000 mm
Device weight	160 kg (including battery weight 33,5 kg)

Note that maximum speed, range and battery charging time depend on conditions. These values are affected, for example, by operating temperature, driving style, road profile and user weight.

9. Wiring diagram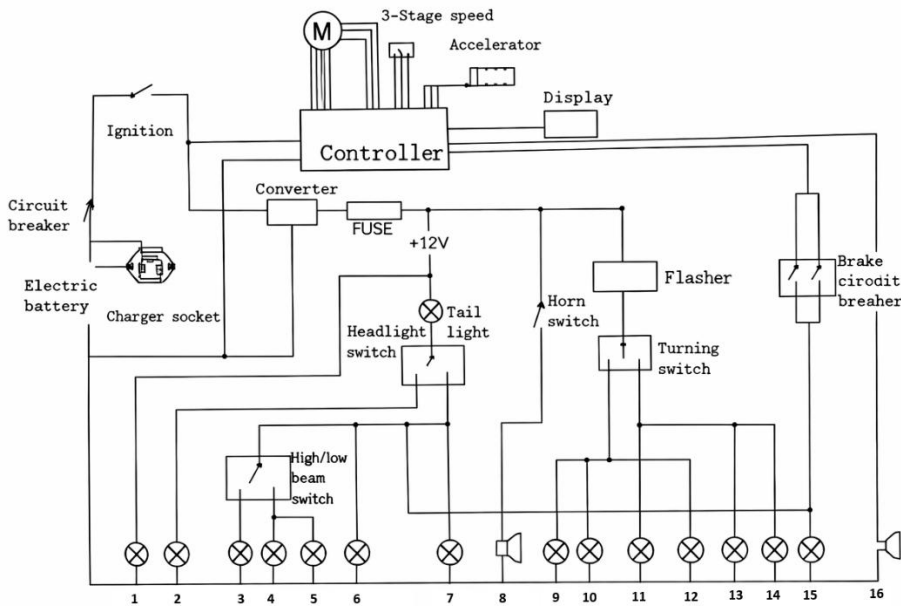

1. Parking lights
2. Lights
3. Low beam
4. High beam
5. High beam indicator
6. Display light
7. License plate light
8. Horn
9. Left turn signal indicator
10. Front left turn signal
11. Front right turn signal
12. Rear left turn signal
13. Rear right turn signal
14. Right turn signal indicator
15. Brake light
16. Reverse buzzer

10. Warranty

Agilon Elan senior mobility scooter is manufactured for durability; warranty periods for defects caused by material or manufacturing faults are listed below.

Part name	Warranty	Covered conditions
Control unit	1 year	Damage not caused by external factors attributable to the user
Frame	1 year	Natural separation of welds, loosening of solder joints, cracks in structural components, deformation or fractures
Brakes	1 year	Failure of the braking system, leakage or breakage of the brake lever
Charging unit	1 year	Failure
Battery	2 years	Failure Capacity reduction as follows 1. within the first 15 months, capacity below 80 % 2. 16-24 months commissioning, capacity below 70 %
Charger	1 year	Does not function normally
Display panel	1 year	Malfunction, display does not function correctly
Tires	3 months	Cracks, fractures or abnormal surface wear