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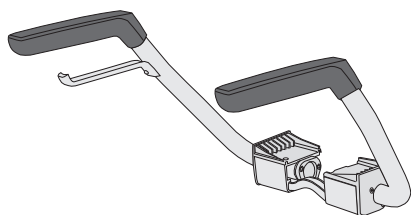
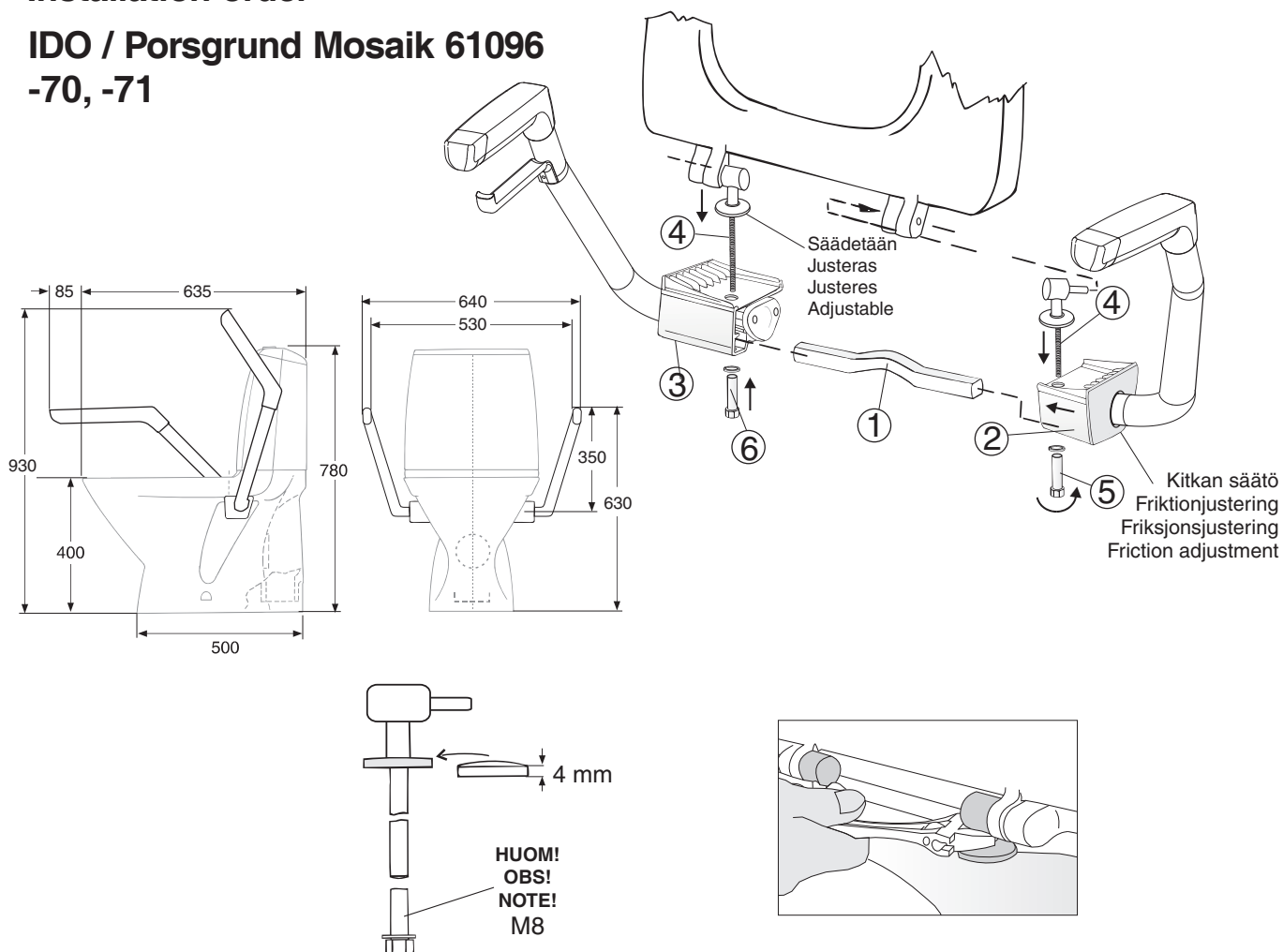
The arm supports and the toilet roll holder are of matt anodized aluminium. The handles have plastic covering. The parts needed for attachment come with the arm supports. The arm supports for the Trevi E model are attached with the hinge bolt of the seat and can be lifted separately. **They fit Trevi E toilet seats -92, -95, -96, -97, -98, -99.** Fasten the WC to the floor before installing the arm supports.

**Kontrollerer fuktspærren når du lager hull til boltene. Take the moisture trap into account when drilling the fixing hole.**

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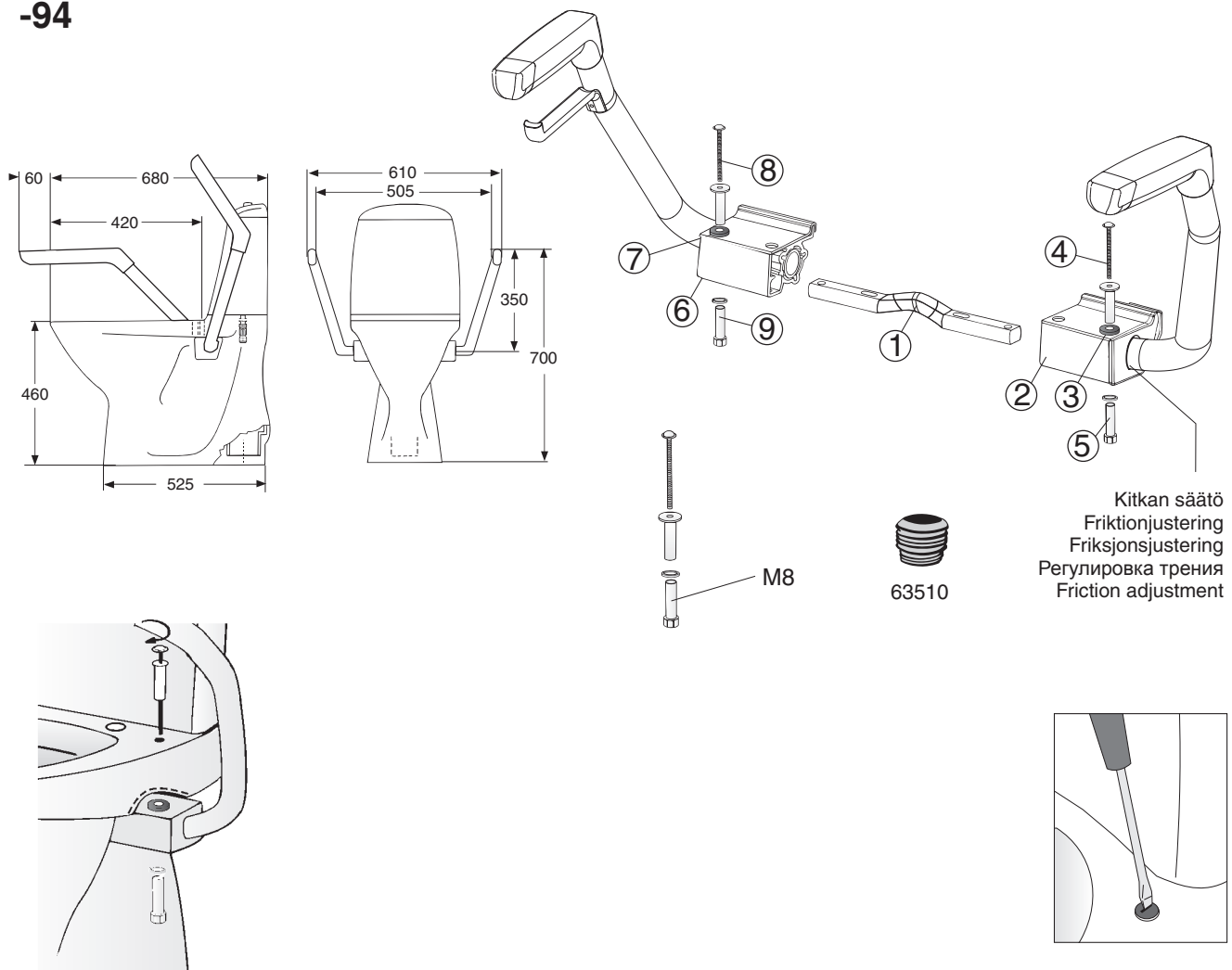
The arm supports and the toilet roll holder are of matt anodized aluminium. The handles have plastic covering. The parts needed for attachment come with the arm supports. The arm supports for the Mosaik model are attached with the hinge bolt of the seat and can be lifted separately.  
**They fit Mosaik toilet seats no. -70 and -71.**  
Fasten the WC to the floor before installing the arm supports.

**Kontroller fuktspærren når du lager hull til boltene.**  
**Take the moisture trap into account when drilling the fixing hole.**

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The arm supports and the toilet roll holder are of matt anodized aluminium. The handles have plastic covering. The parts needed for attachment come with the arm supports. The arm supports for the Trevi E model are attached with special bolts to the seat and can be lifted separately. **They fit Trevi E toilet seats no. -94.** Fasten the WC to the floor before installing the arm supports.

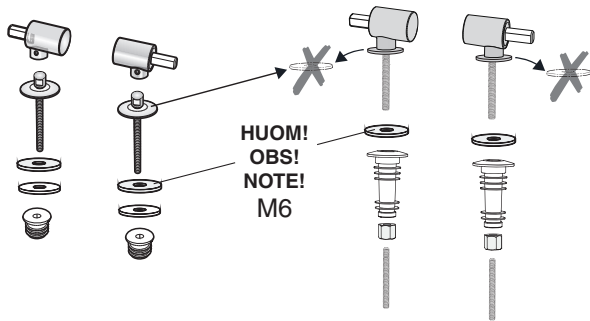
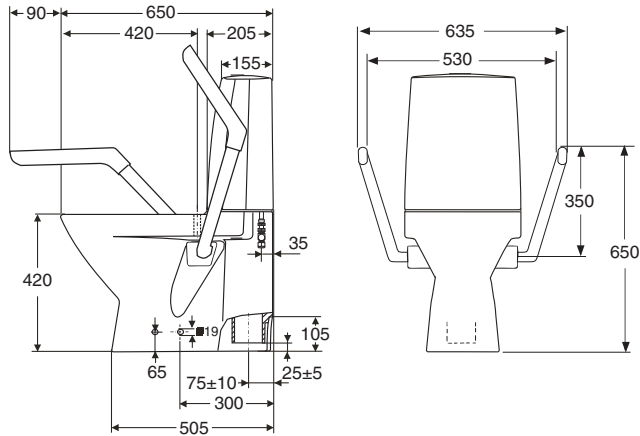
**Kontroller fuktspærren når du lager hull til boltene. Take the moisture trap into account when drilling the fixing hole.**

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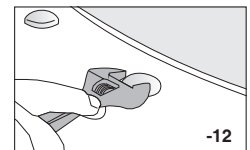
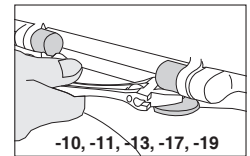
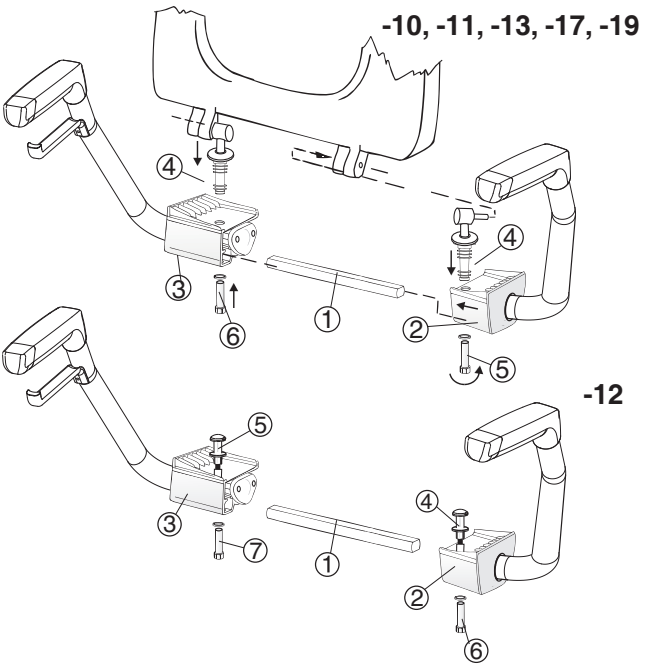
When installing the arm supports replace the stamped washer of the hinge pin with the turned one.

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The arm supports and the toilet roll holder are of matt anodized aluminium. The handles have plastic covering.

The parts needed for attachment come with the arm supports.

Arm supports for the Glow WC models are attached with the hinge bolts of the seat.

Arm supports can be lifted separately.

Fasten the WC to the floor before installing the arm supports.

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### GB

#### WC-bowl with arm supports

Compared to a standard WC-bowl a WC-bowl with arm supports is exposed to considerably greater lateral forces. Therefore we recommend that a WC-bowl with arm supports should be fastened both by screws and also with elastic anti-mould silicone putty.

#### Floor penetration

Floor penetration shall be performed in conformity with the regulations pertaining to floor and wall penetration procedures in sanitary cabins.

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### GB

The product includes a guarantee which is registered to the product at the time of sale in accordance with IDO Kylpyhuone Oy's price-list.

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### GB

The product has been produced according to stringent quality standards. If, however, the product reveals a defect, we ask you in the first place to send a notice of defect to the store from which you bought the product. To facilitate the claim process, please supply the product number and the date of purchase or installation.

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**IDO Badrum Oy Ab**