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EN

Introduction

Thank you for choosing the Paiseec Mobility Scooter.

The Paiseec Mobility Scooter is an outdoor mobility scooter designed specifically and primarily intended for the elderly and mobility-impaired individuals.

We care about your safety, comfort, and ease of use. Please read this manual carefully prior to product use to ensure proper and safe operation.

Please keep this manual stored properly for reference.

The text, illustrations, etc., in this manual may differ slightly from the actual product due to quality improvement or design changes. Our company reserves the right to make product modifications. Your understanding is highly appreciated. If there are any differences between this manual and the actual product, please refer to the actual product.

Friendly Reminder:


Please read the user manual carefully and familiarize yourself with product performance and operation prior to initial use. Please ensure this manual is properly stored.

1. Product information

Package contents

- 1 x Paiseec Mobility Scooter
- 1 x Basket
- 2 x Mobility scooter key
- 1 x Charger
- 1 x User manual
- 1 x Certificate of conformity

Steering system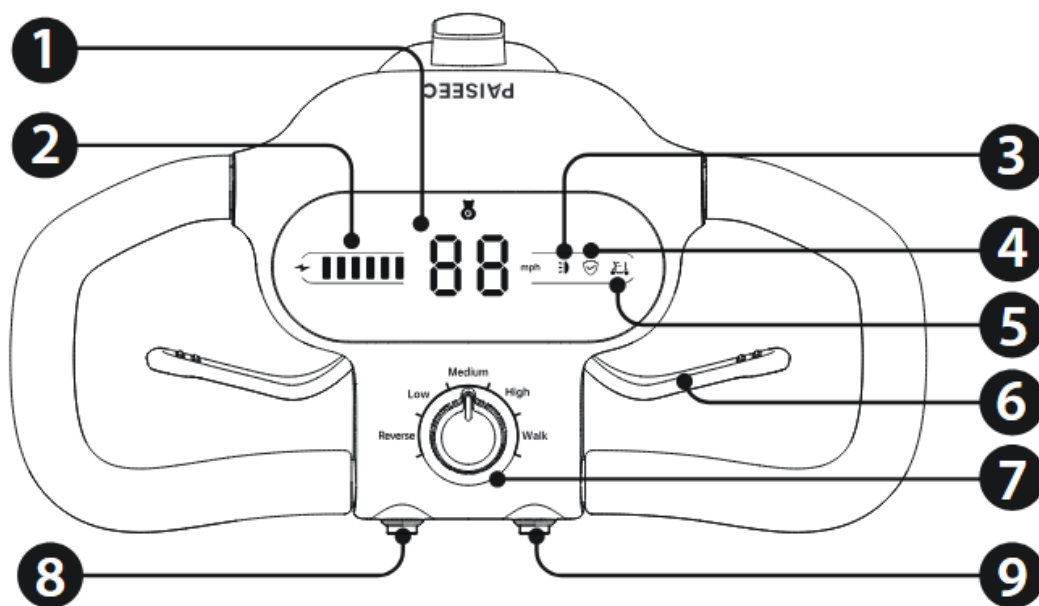

1. Speed display
2. Battery indicator
3. Headlight indicator
4. Safety indicator
5. Riding status
6. Throttle lever
7. Mode knob (Reverse/Low/Medium/High/Walking)
8. Horn switch
9. Light switch


Mobility scooter parts

1. Steering Handlebar
2. Basket
3. Headlight
4. Front bumper
5. Armrest
6. Seat
7. Rear wheel 10"
8. Front wheel 9"
9. Key switch
10. Charging port
11. Storage hook
12. Pole folding mechanism
13. Floor liner
14. Electromagnetic Brake
15. Anti-tip wheel
16. Seat folding handle
17. Battery lock cover

Battery and charger


1. Charger indicator
2. Charger connector
3. Power cord
4. Battery charging port
5. Battery Connector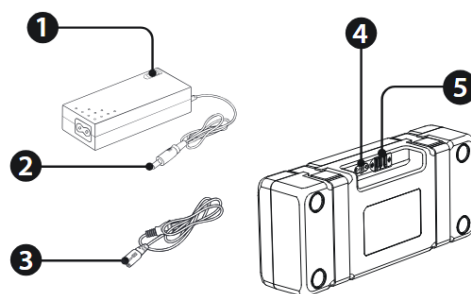

2. Functions


Unfolding the Mobility Scooter

1. Lift the pole to an upright position and then lock the folding mechanism (first secure the folding mechanism upward, then rotate the plastic buckle to the left to lock it into place)
2. Lift the seat and listen for a clicking sound, which indicates that the seat is fully unfolded.
3. Install the basket to the front of the pole.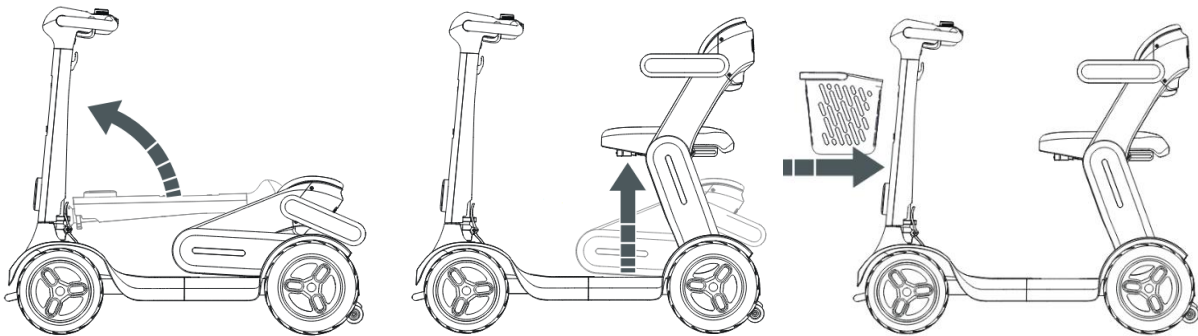

Folding the Mobility Scooter


1. Remove the basket from the mobility scooter.
2. Lower the seat and lift the armrest by pulling the seat folding handle underneath the seat forward and gently lower the seat to fold.
3. Release the pole folding mechanism (rotate the plastic buckle to the right before pulling the folding mechanism down) and lower the pole.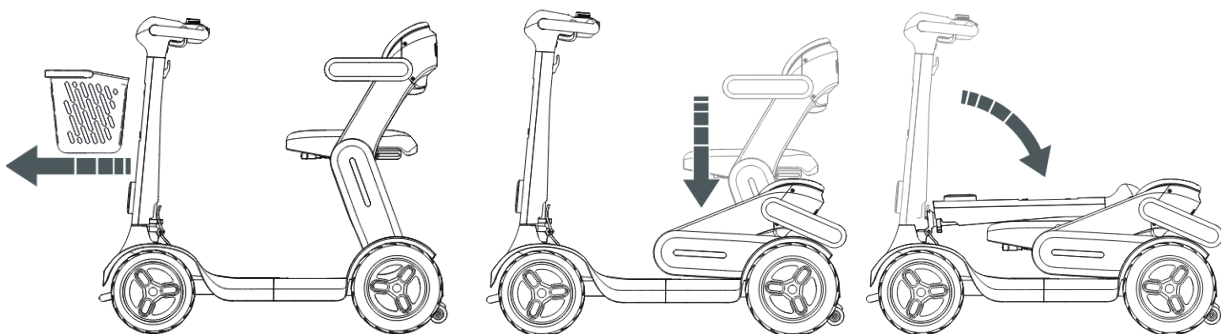

Battery Detachment and Placement

Battery Detachment

Insert the key into the key hole on the back end of the foot rest. Turn the key to the "Unlock" position. Pull the key to lift the battery cover. Pull the handle straight out on top of the battery to take the battery out of the cartridge. Please make sure the battery is taken out vertically to avoid damage to the quick connectors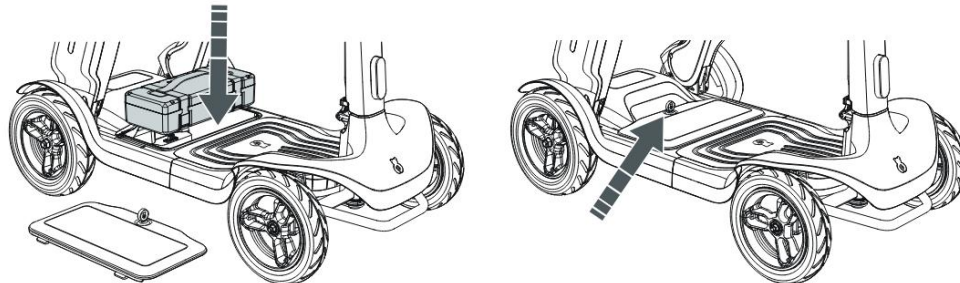

Installation of Battery

Align the connectors on bottom of the battery to the connectors in the battery cartridge. Then put the battery straight down vertically. Put the battery cover back with the key on it. Turn the key to "Lock" position to seal the battery cartridge. Please be sure the cover is sealed, or adjust it to proper position then turn the key to seal.

2. Usage

Inspect following before starting the device:

1. Overall Scooter

Please ensure all folding components of the mobility scooter are correctly assembled and locked, with no parts missing, damaged, or loose.

2. Electromagnetic Brake

Please check whether the brake lever can be pulled down to release the electromagnetic brake for manual pushing.

Please check whether the brake lever can be lifted up to deactivate the electromagnetic brake, which ensures that manual pushing is not possible.

3. Wheels

Please check whether the tires are properly inflated.

Please examine the front and rear wheels for wear, with groove depth equal to or greater than 0.5mm. Please ensure there are no foreign objects lingering.

Please check whether the front or rear wheels are cracked or deformed.

Please check whether there is noticeable wobbling of the front and rear wheels during manual or powered movement.

4. Seat

Please check whether the seat is adjusted properly, with corresponding armrest studs securely fastened, and confirm that the seat is stable.

Please check whether the armrest studs are securely fastened and if there is any wobbling.

If there are any abnormalities in the above parts, please refrain from using and promptly conduct maintenance.


Power on inspections

1. Check whether the battery level and other indicators on the dashboard are displaying normally when the power is on.
2. Please check whether the mobility scooter battery is sufficiently charged.
 - The 6 bars of indicators on the left side of the dashboard indicate the battery level, and the more bars are on, the higher the battery level is.
 - If the rightmost red light on the battery indicator is on, it indicates that the remaining battery is low. Please promptly charge to avoid running out of battery during your ride.
3. Please check whether the headlight switch can be turned on and off normally.
4. Please check whether the mobility scooter horn button is working and whether the horn sound is audible.
5. Please check whether the mode adjustment knob is working correctly, and whether the voice prompts are normal when shifting the knob.
6. Please ensure that the throttle lever is not loose and can be operated normally.
7. Please check whether there are any abnormal sounds when the tires rotate.
8. Please check whether the electromagnetic brake is in the closed position.

If there are any abnormalities in the above parts, please refrain from using and promptly conduct maintenance.

Starting the Mobility Scooter

Insert the key and turn it to the right to start the mobility scooter, the dashboard will light up.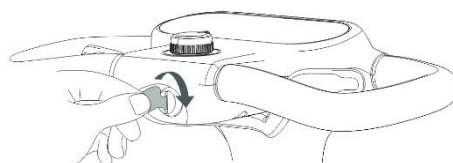


Going forward

1. Put the Mode knob at "Low" speed mode
2. Hold the throttle lever (either on the left or right side) for forward operation.
3. Adjust the speed as needed.

Low: low speed mode, 2mph (3km/h);

Medium: medium speed mode, 4mph (6km/h);



High: high speed mode, 6mph (10km/h).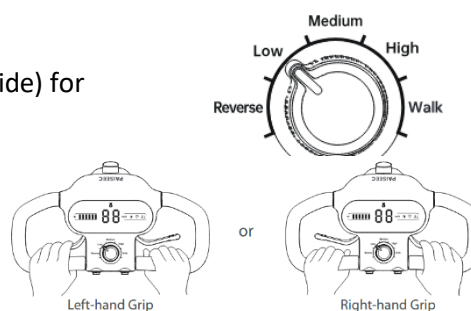

During the initial learning phase, it's recommended to practice at low-speed mode.

Reverse



1. Put the Mode knob in “Reverse”.
2. Hold the throttle lever (either on the left or right side) for reverse operation.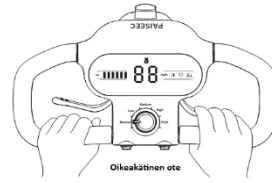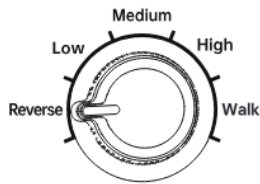


Reverse speed: 1mph (2km/h).

Braking

1. Normal braking: release the throttle lever to stop (whether in forward or reverse mode).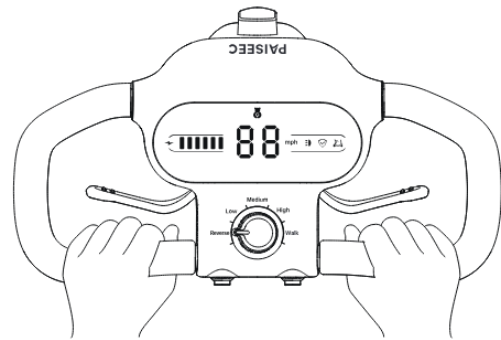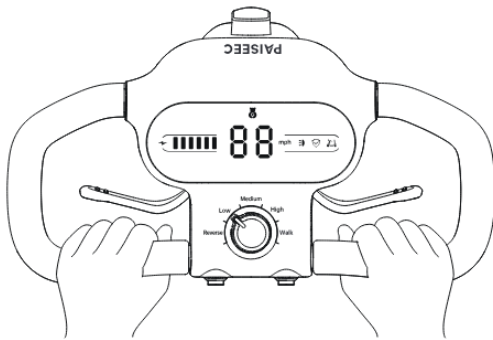

2. Emergency braking: turning the key all the way to the left activates emergency stopping, further reducing the braking distance.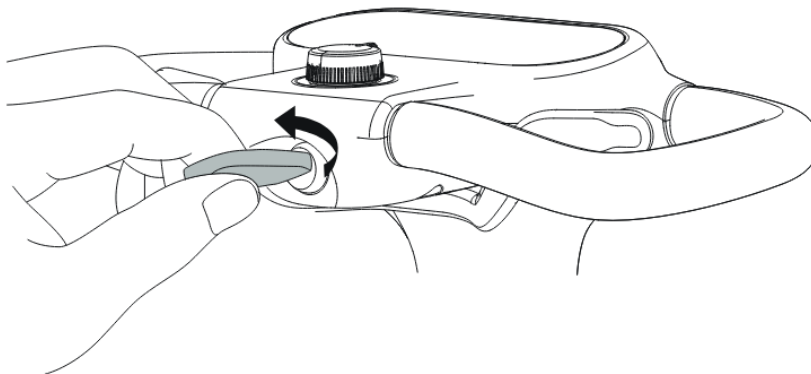


3. We provide braking assistance for users in the following situations:

- Switching mode from “Reverse” to “Low”
- Switching mode from “Low” to “Reverse”.
- Switching mode from “High” to “Walk”.
- Switching mode from “Walk” to “High”.


Turning

1. During riding, turning left or right is completed by turning the steering handlebar.
2. While making turns, reduce speed and perform steering maneuvers slowly. It is recommended for to practice turns multiple times before using on the road.
3. The safety indicator will light up if your speed is excessive while turning, and the intelligent algorithm-based assisted deceleration will activate to reduce rollover risk.
4. Avoid abruptly turning the handlebar at high speeds, as this may cause a rollover.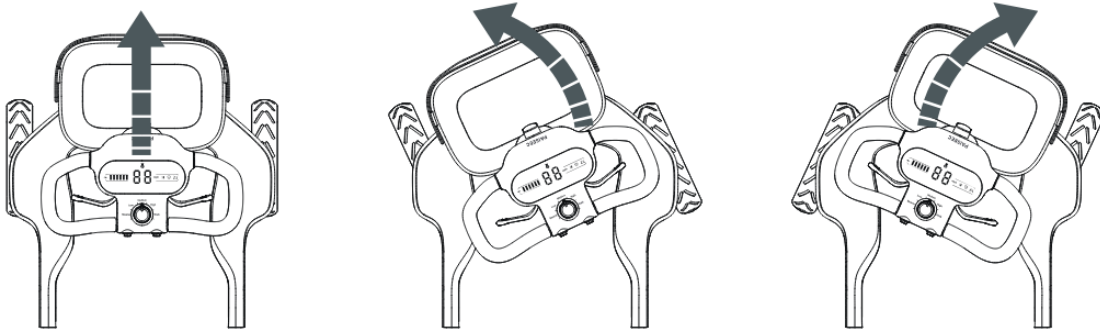

Walking


1. Put the Mode Knob at “Walk” mode.
2. You may walk the mobility scooter by holding the throttle lever (either on the left or right side); steer by turning the handlebar during walking.
3. While walking, releasing the throttle lever will initiate braking and stop the mobility scooter.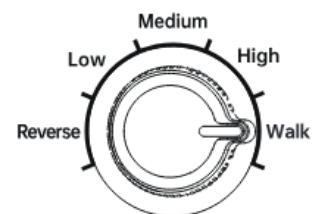

Releasing the electromagnetic break

In case of a malfunction during travel that inhibits normal riding or pushing, you can manually release the electromagnetic brake to walk the mobility scooter forward.

Release method: Pull down the brake handle at the bottom to release the electromagnetic brake

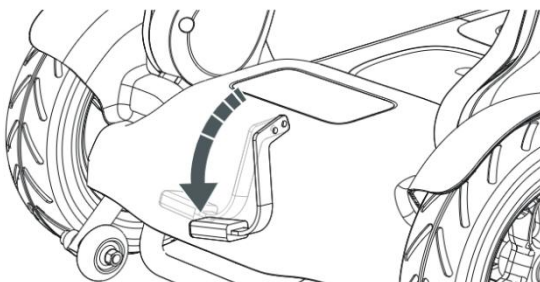Warning

Only release the electromagnetic brake when necessary, as doing so unnecessarily could pose a danger.

Riding Conditions Requirements

1. Avoid ascending steps higher than 2"/5 cm.

2. Refrain from navigating slopes steeper than 8°.

3. Do not drive sideways on slopes steeper than 5°.

4. Do not attempt to cross gaps wider than 4"/10 cm.

5. Do not drive on soft surfaces such as sand, marshes, or snowy and icy roads.

**Warning**

For your safety and the safety of others, please be sure to read and comply with the following safety warnings and relevant laws and regulations.


For your safety, please read the user manual carefully prior to riding.

1. When riding, please carefully observe the surroundings and road conditions. When riding in crowded areas, near walls, obstacles, narrow areas, uneven surfaces or slopes, ride cautiously at low speed.
2. When approaching curbs, steps, slopes, and obstacles, approach them head-on and proceed at low speeds.
3. Avoid coming into contact with the wheels. During travel, do not allow people around you to touch the wheels, as there is a risk of injury.
4. Please do not intentionally release the electromagnetic brake, as it may result in the mobility scooter moving and resulting in an accident.
5. Please mount or dismount from the mobility scooter on a flat and level surface.
6. Please take extra care when riding on slopes. When riding inclines, prioritize safety and adjust the speed to "Low" whenever possible.
7. When parking on a downhill slope the braking distance will increase. Please release the throttle lever in advance and allow sufficient time for stopping.
8. The mobility scooter will appropriately reduce speed as the slope increases to ensure rider safety when riding downhill.
9. Do not lend the mobility scooter to individuals who are not familiar with its operation.
10. Individuals under the age of 16 are strictly prohibited from riding the mobility scooter.
11. It is recommended to wear helmet, goggles, and other protective equipment when riding.
12. The braking distance will increase on slippery roads. Please reduce speed or avoid nonessential travel.
13. Please turn on the headlights during night riding.
14. Riding while intoxicated is prohibited.

Load Requirements

1. The weight of the driver should not exceed 265lbs (120kg).
2. The items stored in the basket should not exceed 8.8lbs (4kg).

Dashboard Indicators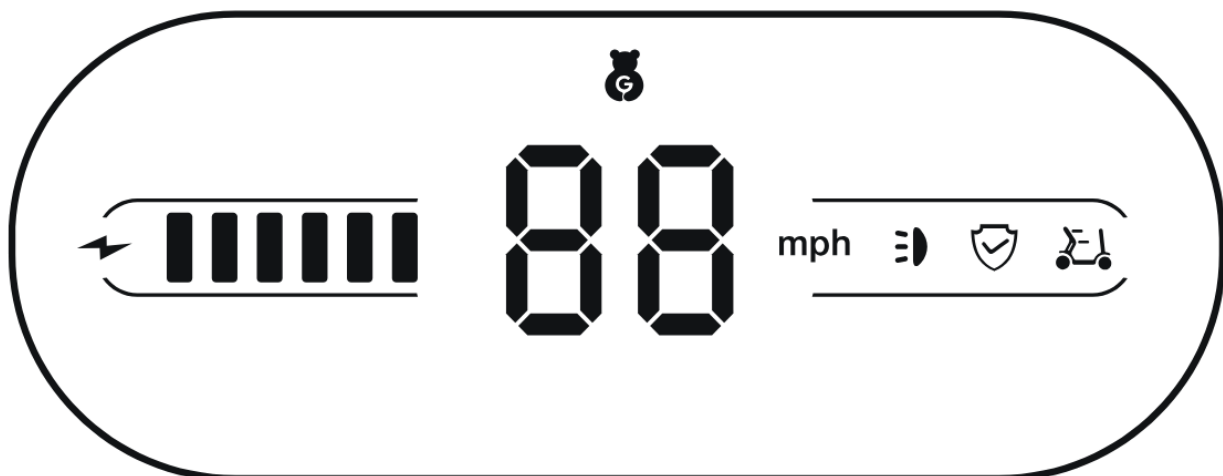


Indicator	Name	Description
	Riding Status Indicator	White: Normal riding or usage. Red: Unable to ride or use. Common situations in which red may appear: 5. Failure to sit on the seat under the “Low-Medium-High” Mode Knob (On-seat sensor is not effective, only applicable to S3). 6. Putting the Mode Knob in “Walk” mode when riding on a steep slope. 7. The indicator light becomes red after the mobility scooter is started and throttle lever toggled mobility scooter; The throttle lever must be released and mobility scooter restarted to be able to use it normally. 8. During the Mode Knob adjustment process, if the Mode Knob adjustment knob is not adjusted to the correct position, the red light will also appear when in-between Mode Knob. After adjusting to the correct position, the red light will turn off.
	Safety Indicator	O-: The “PAi” Safety System is not activated. White: The “PAi” Safety System has been activated to detect road conditions and optimizes mobility scooter operation for a better driving experience.
	Headlight Indicator	O-: The headlight is off. White: The headlight is on.
88	Speed Display	Current mobility scooter speed.
	Battery Indicator	Current remaining battery level.
	Charging Indicator	Lights up during charging.


Voice prompt volume adjustment

The default volume of voice prompt is loud. If you want to turn the volume down or shut it o-, please follow our instructions below. By taking the steps below in a row you can adjust the volume to 100%, 50% and mute.


1. Start the scooter by turning the key switcher to "on"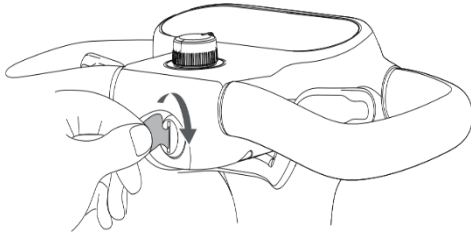


2. Turn the mode knob to "walk" mode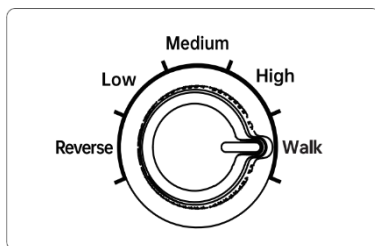

3. Press the "Light" button. By now the headlight is on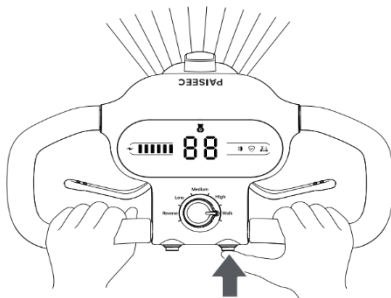


4. Press and hold the "horn" button for 8 seconds. Until hear the voice: volume adjusted to 50%. Please drive with caution.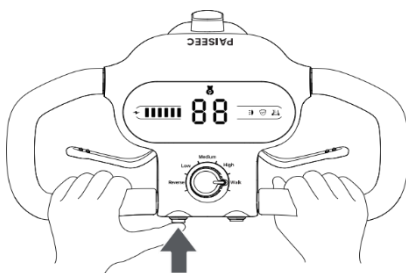

After these steps, the volume will be set to 50%. To mute the voice prompt from 50%, you will need to do these steps all over again. By doing all these steps three times, the volume will be set to the default settings (100% volume).

The volume of horn will not be modified by these operations to make sure the horn will honk when you need it when riding.

4. Battery and charging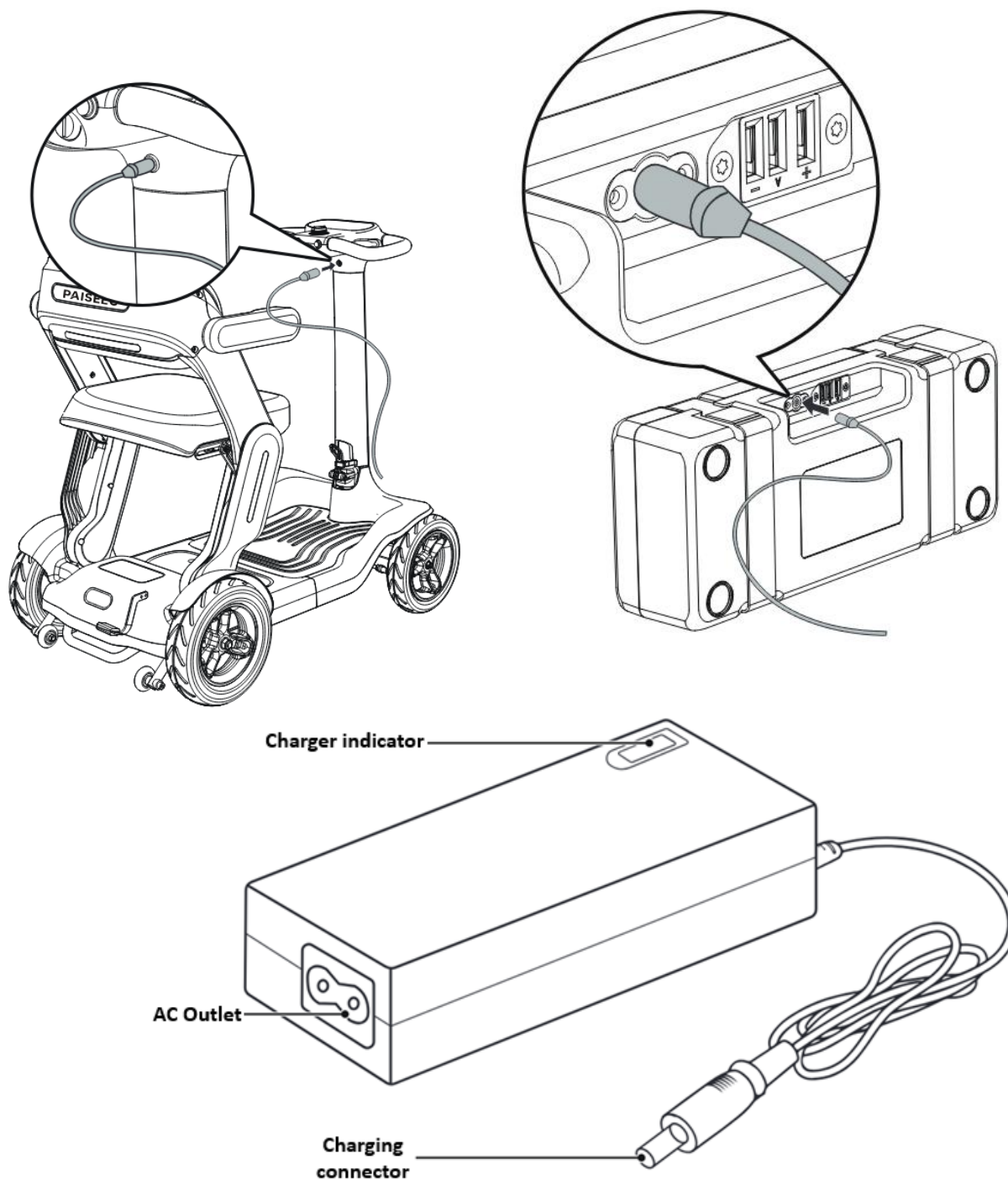

Charger indicator lights

Red: Charging
Green: Fully charged or on standby

8. Power off the mobility scooter and plug the dedicated Paiseec Mobility Scooter charger into the charging socket.
9. Connect the charger: Plug the charger into a socket (110V~220V AC power plug) and connect to the charging port under the front of the mobility scooter.
10. Charging: The indicator (red light) on the charger will be on. At the same time, confirm that the charging indicator on the mobility scooter's dashboard is illuminated, which indicates that charging is in progress.
11. Charging duration: 8 ~ 13 hours (may vary depending on the battery specifications, 7~9h for a 12Ah battery and 11~13h for a 20Ah battery after fully discharging).
12. Charging completion: the charger powers o- automatically when the battery is fully charged. The green light stays lit, indicating that charging is complete.
13. Disconnecting the charger: unplug the charger from the mobility scooter charging port mobility scooter before unplugging the charger from the socket.

Battery Maintenance and Charging Precautions

Please carefully read the instructions in this manual regarding batteries and chargers and follow the instructions provided.

8. Please use the original Paiseec Mobility Scooter charger.
9. Do not disassemble the Paiseec Mobility Scooter battery and charger.
10. Disassembling the battery or charger may lead to res or battery explosions.
11. Disassembling the mobility scooter or charger will void the warranty.
12. Charging environment requirements
 - a. Please charge in an environment within a temperature range of 32-104°F. Charging the mobility scooter outside of this temperature range may result in inability to charge or even damage the mobility scooter or battery.
 - b. Avoid charging at high temperatures: Charging at high temperatures can cause the battery to overheat and cause damage.
 - c. Avoid charging at low temperatures: Charging in temperatures below zero should be avoided to prevent battery overcharge.
 - d. Avoid humid environments: Avoid charging in humid environments to prevent short circuits.
 - e. Charge the battery in a well-ventilated and sufficiently large space (at least twice the area of the mobility scooter) free of flammable gases. If the charging environment contains flammable gases or poor ventilation, there may be an explosion risk or other safety hazards.
13. Charging requirements
 - a. Please charge the mobility scooter from a 110~220V /50~60Hz power outlet.
 - b. Avoid using excessively high or low voltages to charge the mobility scooter, as this can damage the device or pose other dangers.
 - c. After charging is complete, please disconnect the charging connector from the mobility scooter.
 - d. Prolonged insertion of the charging connector into the battery can lead to a decrease in battery quality.
 - e. Mobility scooter batteries typically require 8 to 13 hours of charging to reach maximum capacity. Overcharging will damage the battery and shorten its lifespan.

The Paiseec Mobility Scooter charger will automatically power off after the battery is fully charged. Please use the original charger to avoid battery overcharging.

- f. Do not touch the battery and charger during or after the charge is complete. During the charging process, the temperature of the battery and charger will rise. Do not touch to avoid the risk of heat burns.
 - g. Do not directly touch or wet the interface, nor place it near any metal or other foreign objects. During the charging process, such actions pose a risk of short circuits, which may cause personal injury and damage to the mobility scooter and parts.
 - h. Do not use extension cords with the charger as there is risk of electric shock, short circuit, and fire.
 - i. Do not pull the power cord excessively hard as there is risk of electric shock, short circuit, and fire.
 - j. Do not handle the battery with wet hands as there is risk of electric shock.
 - k. Do not charge the battery during brake release as there is risk of accident due to mobility scooter movement.
14. Storage requirements for the mobility scooter and charger.
- a. Keep the mobility scooter and battery away from heat sources to avoid explosions or other accidents.
 - b. Do not let the mobility scooter or battery come into contact with water. Do not immerse the battery in water to avoid short circuits.
 - c. Keep the battery and charger dry to avoid short circuits or other malfunctions during subsequent charging.
 - d. Prevent the battery or charger from any collision (impact, dropping or force, etc.) to avoid other accidents arising from damage to internal parts.
 - e. If the battery is damaged externally or its function is found to be abnormal, stop using or charging immediately and contact customer service.
 - f. Be sure to close the cover to avoid water ingress or other damage when the battery charging port is not in use.
 - g. Regularly check the charger and battery: Regularly check the charger and battery for damage or wear. If any issues are discovered, please replace the charger or battery immediately.
 - h. If not in use in 30 days, the scooter should be charged to 80% power (five bars on indicator), and should be stored in cool and dry places. Long term storage requests the scooter for at least 4 hours charge in 30 days and to make sure there is 50% power (3 bars on indicator).
 - i. Charge the scooter every 3 months to 100% to lengthen the span life of battery.
 - j. Charge the scooter in 7 days if the battery indicator has one bar remaining after driving. Or the battery may be damaged of over-discharge.
 - k. Note: If not been charged in time, the battery may sustain irreversible damage of over-discharge. Battery failure causing by over-discharging is not included in warranty.

5. Care and maintenance

To maintain the mobility scooter's optimal performance, reduce the occurrence of malfunctions, and ensure your safe and comfortable riding experience during everyday usage, please pay attention to the following:

7. Carefully read the safety operation instructions and understand the meaning of each item. The Paiseec Mobility Scooter controller is a critical component of the entire mobility scooter. Do not attempt to open, repair or perform actions that alter the controller. Regularly wipe the controller surface with a dry and soft cloth to remove dirt. It is strictly prohibited to use the controller in environments with excessive temperature or high humidity.
8. Do not attempt to open, repair or manipulate the internal structure of the motor, as doing so can damage the motor and void the warranty. Prolonged motor overloading or blockage is strictly prohibited. You should regularly clean the surface of the motor, especially the dirt on the motor lead-out wires. Do not expose the motor to water. Crossing water with a depth of more than 5cm may lead to water entering the motor, which may result in permanent motor damage.
9. Do not use chemicals like gasoline to clean the housing and other parts of the Paiseec Mobility Scooter. Instead, use a dampened cloth to wipe. Additionally, carry out regular inspections and maintenance on the mobility scooter.
10. Regularly check if charging is needed and inspect the brakes for any malfunction before riding. Periodically examine the tightness of the lever's fixing screws and ensure the lever is in the neutral position. Keep the electrical system dry. Regularly assess the battery's condition and wipe the battery connector if necessary.
11. If the mobility scooter isn't used for an extended period, charge it once a month. Regularly check the insulation of the wires and the quality of plug connections. Check whether there is any wobbling of the front and rear tires.
12. In case of any abnormalities, please visit a dealership where professional technicians can perform a comprehensive inspection mobility scooter.

6. Technical specifications

Scooter performance	Max Speed	10 km/h
	Maximum Climbing Angle	8°
	Maximum Parking Angle	15°
	Maximum Obstacle	5 cm
	Crossing Height (Stepped)	7 cm
	Maximum Obstacle Crossing	-15-40 °C
Battery specifications	Battery Specifications	37V 12Ah lithium battery
	Charger Specifications	42V 1.5A
	Battery Detachment	Supported
Mobility scooter specification	Expanded Dimensions (LxWxH)	109×55×88 cm, including basket 99×55×88 cm, without basket
	Folded Dimensions (LxWxH)	99×55×43 cm, without basket
	Mobility Scooter Weight	32 kg (including battery)
	Rear Wheels	9" pneumatic tires
	Front Wheels	10" pneumatic tires
	Ground clearance	7 cm
Seat	Seat size	40×35 cm

Electronic system	Drive system	PAi-safety system
	Motor specification	250 W brushless motor
	Brake system	dual brake system: electromagnetic brake & electronic brake
	Unlocking method	Mechanical key
Storage space	Hook	Supported
	Basket	Supported
Charger specification	Input power voltage	100-240 V, 50/60 Hz
	Rated output voltage	42V
	Rated output current	1,5 A
	Operating temperature	0-40 °C
	Charging time	8-10 hours
Mobility scooter load	Load weight	User weight 120 kg Max climbing angle: 8° Max speed on flat road: 10 km / h Max speed on 8° slope: 6 km/h
		User weight 100 kg Max speed on 8° slope: 6 km/h
		User weight 75 kg Max speed on 8° slope: 10 km / h

Note: This data is only for reference of the speed in different load and slope conditions, which effected by conditions of pavement, wetness, gradient, battery power percentage, and load. The real speed may vary in practice.

6. Common malfunctions and trouble shooting

Malfunctions	Cause of Malfunction	Troubleshooting
The mobility scooter cannot be powered on after turning the key switch	1. Battery voltage too low 2. Key switch malfunction	1. Fully charge the battery 2. Contact dealership to replace the malfunctioning component
Throttle lever is unresponsive	1. Battery voltage too low 2. Electromagnetic brake switch not closed 3. Mode Knob switch misaligned 4. On-seat sensor deactivated.	1. Fully charge the battery 2. Turn of the electromagnetic brake switch 3. Toggle the Mode Knob switch properly 4. Contact dealership to replace the malfunctioning component
Insufficient cruise distance after charging	1. Insufficient charging or charger malfunction 2. Battery aging or damage	1. Charge the battery sufficiently or replace the charger 2. Contact dealership to check if the battery is damaged and replace the malfunctioning component
Charger not charging	1. The charger socket comes of or the plug connection is not secure 2. AC power supply malfunction 3. Battery malfunction	1. Fasten the socket and the connector 2. Check the power supply or adjust the transformer 3. Contact dealership to check if the battery is damaged and replace the malfunctioning component
Power interruption during riding	1. Controller protection activated 2. Battery box thermal protection activated	1. Turn the mobility scooter of and on again 2. Resume driving after the motor cools down
Other malfunctions	Other malfunctions not covered by the guidance above	Contact the supplier or service station. Do not disassemble the mobility scooter yourself.

7. Warranty terms

In order to ensure customer's interests and provide high quality after-sales service, our company will offer free replacement parts or warranty service for damages caused by product quality issues or material problems during the warranty period.

Part name	Warranty	Warranty Scenario
Mobility scooter	1 year	Damage caused by factors other than human-induced external forces
Frame	1 year	Natural weld separation, desoldering, cracks at key components, distortion, and fractures
Motor	1 year	Magnet detachment, coil burnout
Gearbox	1 year	Cracked enclosure, oil leakage
Battery	12 months / 18 months	Cracked lead enclosure, leakage, less than 60% capacity; 12 months warranty for lead-acid batteries 18 months warranty for lithium batteries
Charger	1 year	Unable to charge, cannot be operated normally
Controller	1 year	Mobility scooter cannot be operated normally
Dashboard	1 year	Malfunction, unable to display properly
Tires	6 months	Cracking, fracture, and natural tire burst

Warranty Exclusions

The warranty service does not cover the following situations, the repairs and related service are to be paid for accordingly:

1. Exceeding the specified service period and scope stipulated in the After-sales Warranty Policy of Paiseec Products.
2. Incorrect usage, operation, or maintenance of the Paiseec electric product according to the provided instructions.
3. Damage resulting from exposure to rainwater or smoke, immersion in ice or snow, or corrosion caused by substances like pharmaceuticals or chemical products, as well as damage or malfunctions caused by natural disasters.
4. Regular wear and tear is not covered by the warranty. Malfunctions or damage caused by force majeure (including but not limited to earthquakes, typhoons, fires, floods, social incidents, mass incidents, violent crimes, etc.).
5. Not complying with the maintenance suggestions of the battery which led to the damage of battery. Not charging the scooter for more than 30 days. (Note: charge the scooter to 80% power(five bars on the indicator) and store the scooter in places that are cool and dry. Long term storage requests 4 hours charge in at most 30 days to keep the power 50%(three bars on indicator)).
6. Repairs, modifications, disassembly, or damage to the overall product or components performed outside of authorized Paiseec electric mobility scooters service centers.
7. Damage to the components of the Paiseec electric mobility scooter due to the use of non-original parts or unauthorized alterations to circuits and wiring configurations.
8. Product malfunctions or damage caused by human factors such as collisions, falls, overloading, or speeding during riding.

8. Warnings and precautions

Safety has been thoroughly considered during the development and design of the Paiseec Mobility Scooter and extensive safety inspections conducted during production. However, incorrect use of the product or disregard for preventative measures may cause damage or injury to the mobility scooter itself and to individuals.

The Paiseec Mobility Scooter is a new type of mobility scooter. Please use only after thoroughly reading and understanding the contents of this manual.

It is suitable for general paved roads and indoor use. According to Road Traffic Safety Law, using this product does not require a driver's license.

Warnings

When using the mobility scooter, please take note of the following:

1. The mobility scooter is intended for short-distance travel and must not be used on highways or other dangerous roads. Use the scooter in accordance with traffic regulations to prevent accidents.

2. Wear a helmet while driving and follow local safety regulations. Ensure that you are familiar with and comply with local laws and regulations to avoid violating traffic rules. Violations may result in legal liability.
3. The mobility scooter may not be suitable for individuals with certain medical conditions, such as cardiovascular diseases, or for pregnant individuals. Consult a healthcare professional before use to ensure that your health condition allows safe operation. The manufacturer, importer, or distributor is not responsible for personal injuries resulting from individual health conditions.
4. Pay attention to your surroundings and the movement of other vehicles when using the device. Remain alert at all times to avoid accidents. If an accident occurs due to lack of awareness, you are responsible for any resulting damages.
5. Do not use the mobility scooter for purposes other than its intended use, such as transporting goods or for illegal activities. If an accident occurs due to violation of laws or improper use, the manufacturer is not responsible.
6. Do not rent or lend the mobility scooter to others. If renting or lending results in an accident, the manufacturer, importer, or distributor is not responsible.
7. We recommend that you use the mobility scooter in accordance with the above disclaimers, take responsibility for your own safety, and comply with local laws and regulations.

Precautions

Always observe the following when using the device:

1. Before using the mobility scooter, ensure that you fully understand its operation and technical features. Read the user manual carefully and follow the instructions. If you are unsure, seek assistance from a qualified person.
2. Prioritize traffic safety and follow traffic rules to avoid accidents. Obey traffic lights and signs, yield to pedestrians and other vehicles, observe speed limits, and follow road and lane directions. Take road and weather conditions into account and avoid using the scooter in rain, snow, or on slippery surfaces.
3. Do not operate the mobility scooter on lanes intended for motor vehicles. Maintain a safe distance and appropriate speed, and monitor traffic both ahead and behind you.
4. Do not use a mobile phone or other electronic devices while driving, as they may cause distraction and lead to accidents. Focus on driving and avoid activities such as listening to music, answering calls, dialing numbers, or typing messages. If you need to use a device, stop completely first.
5. Do not operate the mobility scooter under the influence of alcohol, drugs, or medications that impair driving or reaction ability, as this reduces judgment and reaction time and may lead to traffic accidents. Use alternative means of transportation if you are impaired.
6. Avoid using the mobility scooter in darkness or low-light conditions. Prefer well-lit areas. If driving in the dark, use appropriate additional lighting and reflectors to improve safety. Stay aware of your surroundings and other road users.
7. We hope these precautions help you use the mobility scooter safely and comfortably. If you have any questions or feedback, please feel free to contact us.
8. We reserve the right to the final interpretation of this user manual. Any printing errors or product improvements will be explained in future versions of the manual. We may introduce new products at any time to meet market needs. In case of discrepancies between the descriptions in the manual and the actual product, the actual product and its supplementary instructions shall prevail.

9. Product warranty card

Dear user,

Thank you for purchasing our company’s product. We will provide you with warranty services as repairing, replacement and return.

Please fill this card carefully, attach the original receipt or online-order details and keep it properly for future purposes.

Owner’s name:

Email:

Shipping address:

Product model:

Serial number:

Place of purchase:

Sellers contact details:

Receipt / order details:

Maintenance records

Service	Fault description	Fault noticed date	Repair date